



Ultrasonic Cleaner Quick Reference Guide

Instructions

1. Place cleaning solution in unit before turning it on.
2. Once the unit is plugged in, the clock will light up and remain lit.

FAQ

1. **Why is the unit loud?** *The unit has a buzzing sound. This means it is cleaning!*
2. **How do I clean the outside of my unit?** *You can use Windex or vinegar to wipe the unit clean. Rinse well when using vinegar.*
3. **Why should I use the de-gas function?** *This allows any extra air bubbles to get out of the water which in turn allows for better cleaning.*
4. **Where is the off switch?** *There is no off switch. It works like your coffee pot. The clock is always on, but the coffee isn't always brewing. It's the same with this unit. It is only 'on' when a cycle is running.*
5. **The heat light isn't on.** *The heat indicator light will only come on when the water is below the desired temperature (Ex. If the heat is set to 80, it will only illuminate if the water is cooler than 80).*
6. **Is it necessary to use the heat function?** *No! The unit will clean effectively without heat. This function will only increase the speed of cleaning.*