

WARNING!

- * Do not look directly at the LED or light guide while the unit is on
- * Do not immerse unit in water or any other liquid
- * Use only for curing dental materials
- * Do not use around any flammable liquids or gasses
- * There are no user serviceable parts inside. DO NOT OPEN THE LIGHT.
- * The wand's casing may get hot after prolonged use. This is normal. If too hot for comfort allow unit to cool for a few minutes.

United States law limits the sale of this device only to dental professionals for use as directed in this manual.

Warranty

First Medica warrants that the Ultra Blast™ shall be free of defects in material and workmanship for a period of two years after the initial sale.



This warranty does NOT cover damage resulting from using the Ultra Blast in a manner inconsistent with instructions found in this manual. The warranty does NOT cover damage due to physical abuse of the unit nor does it cover breakage of the fiber optic light guide.

THIS WARRANTY IS THE SOLE AND COMPLETE WARRANTY OF THE ULTRA BLAST. FIRST MEDICA SHALL NOT BE LIABLE FOR INCIDENTAL OR CONSEQUENTIAL DAMAGES RESULTING FROM THE FAILURE OF THE ULTRA BLAST. IT IS THE OWNER'S RESPONSIBILITY TO TEST THE ULTRA BLAST'S SUITABILITY FOR USE WITH ANY PARTICULAR LIGHT-ACTIVATED MATERIAL.

For warranty service send the unit to:

Ultra Blast Service
First Medica
13030 Homer Smith Rd.
Piedmont, SD 57769
customerservice@firstmedica.com
(800) 777-7072



071709

9

Package Contents



- A - Light Wand
- B - Charging base
- C - Wall charger
- D - Light shield
- E - Fiber optic Tip

2

Care of the Ultra Blast

The Ultra Blast can be cleaned using *Medica DC-10*, or other similar disinfectant cleaners.

The fiber optic glass tip can be sterilized in an autoclave or *Medica-28 Plus* activated glutaraldehyde, or similar glutaraldehyde sterilants.

To order *Medica DC-10* or *Medica-28 Plus* contact First Medica at customerservice@firstmedica.com or call (800) 777-7072 x3

Warning!

- * Do NOT immerse or spray the curing light or charging base in/with any liquid.
- * Do NOT use solvents, phenol or butyl based cleaners. These may damage the finish and will void the warranty.
- * Do NOT use abrasive cleaners such as Comet or Ajax as they will scratch the finish and void the warranty

7

Operation

Operation of the Ultra Blast is very simple. The top button (A) turns the unit off and on. If the light is left without use for 5 minutes it will power down to preserve battery life. Pressing (A) briefly will wake the unit and the small LED by the "0" will illuminate.

Three timer settings are available, 5, 10 and 20 seconds. Times can be selected by pressing button (B) which will cycle through the three options. A small LED next to the time will indicate the selection. A beeper will sound when the unit is turned on, every five seconds and when it shuts off. A curing cycle can be stopped at any time by pressing button (A).

There is no "off" button. The unit will go into a sleep mode after three minutes of inactivity.



4

Operation (cont)

The glass tip can be removed and autoclaved, chemically cleaned or sterilized in glutaraldehyde. Additional tips can be ordered from your dental dealer if needed.

The audible beep of the Ultra Blast can be turned off or left on depending on the user's needs. To toggle the beep off or on press both the on/off and timer setting buttons simultaneously for 3 seconds.

The Ultra Blast can be used continuously with no constraints on the number of cycles before the light must be allowed to cool. The light may become noticeably warm after prolonged use. This is normal as the case serves as a heat sink for the LED. Discontinue use and allow the light to cool if the front end of the case becomes uncomfortably hot.

THERE ARE NO USER SERVICEABLE PARTS IN THE LIGHT. UNSCREWING OR OPENING THE LIGHT WILL VOID THE WARRANTY!

5